




SEER 1000 Download Tool

The SEER 1000 Download tool is a Service tool that has the ability to download and store data from the SEER 1000 recorder in a gtm file format so that those can be imported into CardioDay afterwards.

In order to download data from the SEER 1000 perform the following steps:

- 1) Open the SEER 1000 Download Tool
- 2) Enter the path where the gtm file should be stored in field that states *Enter existing path here*.

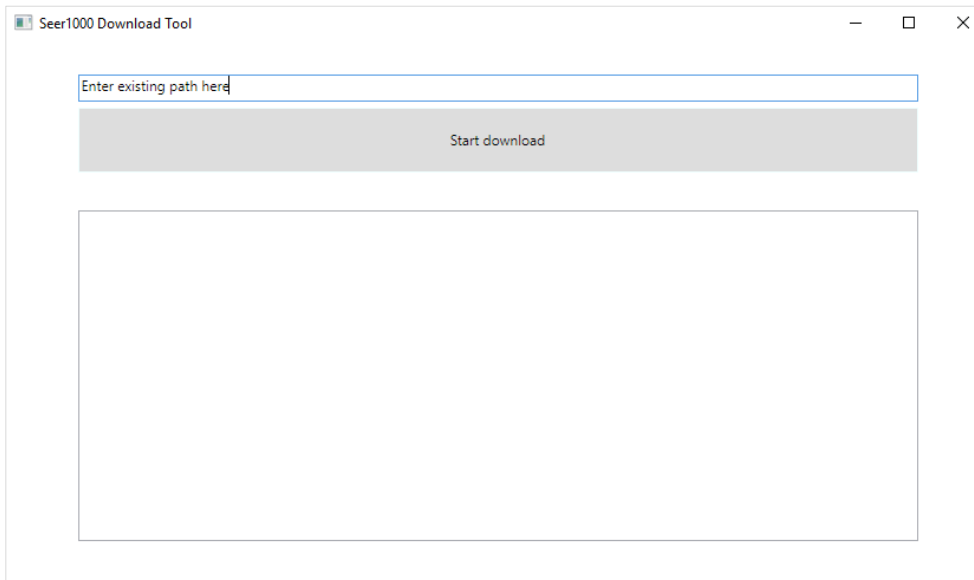

- 3) Make sure that exactly **one** SEER 1000 recorder is connected via USB to the PC.
- 4) Make sure that no CardioDay instance is running on the same PC.
- 5) Start the download process by clicking on **Start download**
- 6) Wait until the message **download ended** is displayed in the window. Note that this can take a few minutes when downloading a 7 day recording.
- 7) Go to the entered directory path and check for the gtm file.

