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HeartX[®] Viewer

Revision 09 DE

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Licensed under The MIT License

Permission is hereby granted, free of charge, to any person obtaining a copy of this software and associated documentation files (the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

React

<https://github.com/facebook/react>

Copyright (c) Meta Platforms, Inc. and affiliates.

Create-React-App

<https://github.com/facebook/create-react-app/tree/master/packages/react-scripts>

Copyright (c) Meta Platforms, Inc. and affiliates.

React-Dom

<https://www.npmjs.com/package/react-dom>

Copyright (c) Meta Platforms, Inc. and affiliates.

http-proxy-middleware

<https://github.com/chimurai/http-proxy-middleware>

Copyright (c) 2015 Steven Chim

Material UI

<https://github.com/mui>

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Copyright (c) 2014 Call-Em-All

Axios

<https://github.com/axios/axios>

Copyright (c) 2014-present Matt Zabriskie & Collaborators

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<https://github.com/zloirock/core-js>
Copyright (c) 2014-2023 Denis Pushkarev

uuid

<https://github.com/uuidjs/uuid>
Copyright (c) 2010-2020 Robert Kieffer and other contributors

dom-to-image

<https://github.com/tsayen/dom-to-image>
Copyright 2015 Anatolii Saienko
Copyright 2012 Paul Bakaus

html2canvas

<https://github.com/niklasvh/html2canvas/issues>

Lodash

<https://github.com/lodash/lodash>
Copyright JS Foundation and other contributors <<https://js.foundation/>>

Based on Underscore.js, copyright Jeremy Ashkenas,
DocumentCloud and Investigative Reporters & Editors <<http://underscorejs.org/>>

This software consists of voluntary contributions made by many
individuals. For exact contribution history, see the revision history
available at <https://github.com/lodash/lodash>

The following license applies to all parts of this software except
as
documented below:

Permission is hereby granted, free of charge, to any person ob-
taining a copy of this software and associated documentation files

(the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

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Licensed under The MIT License

Permission is hereby granted, free of charge, to any person obtaining a copy of this software and associated documentation files (the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall

be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

babel

<https://github.com/python-babel/babel>
Babel License (BSD-3-Clause license)

matplotlib

<https://github.com/matplotlib/matplotlib>
MDT license

reportlab

The ReportLab Toolkit. An Open Source Python library for generating PDFs and graphics.

BSD license

Copyright (c) 2000-2018, ReportLab Inc.

numpy

<https://github.com/numpy/numpy>
BSD-3-Clause Source License
Copyright (c) 2005-2023, NumPy Developers.

The following license applies to all parts of this software

except as documented below:

Permission is hereby granted, free of charge, to any person obtaining a copy of this software and associated documentation files (the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

urllib3

<https://github.com/urllib3/urllib3>

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Copyright (c) 2008-2020 Andrey Petrov and contributors.

azure-ai-ml

<https://github.com/Azure/azure-sdk-for-python>

MIT

h5py

<https://github.com/h5py/h5py>

BSD 3-Clause "New" or "Revised" License

Copyright (c) 2008 Andrew Collette and contributors

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Licensed under The MIT License

Permission is hereby granted, free of charge, to any person obtaining a copy of this software and associated documentation files (the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

urllib3

<https://github.com/urllib3/urllib3>

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Copyright (c) 2008-2020 Andrey Petrov and contributors.

NeuroKit2

<https://github.com/neuropsychology/NeuroKit>

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Copyright (c) 2020, Dominique Makowski

Reportlab

The ReportLab Toolkit. An Open Source Python library for generating PDFs and graphics.

BSD license

Copyright (c) 2000-2018, ReportLab Inc.

Requests

<https://github.com/psf/requests>

Apache License 2.0

matplotlib

<https://github.com/matplotlib/matplotlib>

MDT license

h5py

<https://github.com/h5py/h5py>

BSD 3-Clause "New" or "Revised" License
Copyright (c) 2008 Andrew Collette and contributors

torch

<https://github.com/pytorch/pytorch>
BSD-3-Clause License
Copyright (c) 2016- Facebook, Inc (Adam Paszke)
Copyright (c) 2014- Facebook, Inc (Soumith Chintala)
Copyright (c) 2011-2014 Idiap Research Institute (Ronan Collobert)
Copyright (c) 2012-2014 Deepmind Technologies (Koray Kavukcuoglu)
Copyright (c) 2011-2012 NEC Laboratories America (Koray Kavukcuoglu)
Copyright (c) 2011-2013 NYU (Clement Farabet)
Copyright (c) 2006-2010 NEC Laboratories America (Ronan Collobert, Leon Bottou, Iain Melvin, Jason Weston)
Copyright (c) 2006 Idiap Research Institute (Samy Bengio)
Copyright (c) 2001-2004 Idiap Research Institute (Ronan Collobert, Samy Bengio, Johnny Mariethoz)

pyphen

<https://github.com/Kozea/Pyphen>
GPL 2.0+/LGPL 2.1+/MPL 1.1 tri-license

Flask

<https://github.com/pallets/flask>

BSD-3-Clause License
Copyright 2010 Pallets

Tensorflow

<https://github.com/tensorflow/tensorflow>
Apache License 2.0

scipy

<https://github.com/scipy/scipy>

BSD-3-Clause License
Copyright (c) 2001-2002 Enthought, Inc. 2003-2023, SciPy Developers.

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NumPy

<https://github.com/numpy/numpy>

BSD-3-Clause Source License
Copyright (c) 2005-2023, NumPy Developers.

The following license applies to all parts of this software except as documented below:

Permission is hereby granted, free of charge, to any person obtaining a copy of this software and associated documentation files (the "Software"), to deal in the Software without restriction, including without limitation the rights to use, copy, modify, merge, publish, distribute, sublicense, and/or sell copies of the Software, and to permit persons to whom the Software is furnished to do so, subject to the following conditions:

The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.

THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NONINFRINGEMENT. IN NO EVENT SHALL THE AUTHORS OR COPYRIGHT HOLDERS BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.

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PTB-XL

By accessing and using the HeartX Analyzer project, you agree to be bound by the terms and conditions of the Creative Commons Attribution 4.0 International Public License ("Public License"). If this Public License is interpreted as a contract, you are granted the Licensed Rights in consideration of your acceptance of these terms and conditions. The Licensor grants you these rights in consideration of the benefits the Licensor receives from making the Licensed Material available under these terms and conditions.

[Creative Commons Attribution 4.0 International Public License](https://creativecommons.org/licenses/by/4.0/)

Wagner, P., Strodthoff, N., Bousseljot, R., Samek, W., & Schaeffter, T. (2022). PTB-XL, a large publicly available electrocardiography dataset (version 1.0.3). PhysioNet.
<https://doi.org/10.13026/kfzx-aw45>.

Long Term AF:

[Open Data Commons Attribution License v1.0](#)

Petrutiu S, Sahakian AV, Swiryn S. Abrupt changes in fibrillatory wave characteristics at the termination of paroxysmal atrial fibrillation in humans. *Europace* 9:466-470 (2007).

European ST-T Database:

[Open Data Commons Attribution License v1.0](#)

Taddei A, Distanto G, Emdin M, Pisani P, Moody GB, Zeelenberg C, Marchesi C. The European ST-T Database: standard for evaluating systems for the analysis of ST-T changes in ambulatory electrocardiography. *European Heart Journal* 13: 1164-1172 (1992).

MIT - BIH Malignant Ventricular Ectopy Database:

[Open Data Commons Attribution License v1.0](#)

Greenwald SD. Development and analysis of a ventricular fibrillation detector. M.S. thesis, MIT Dept. of Electrical Engineering and Computer Science, 1986.

MIT-BIH Atrial Fibrillation Database:

[Open Data Commons Attribution License v1.0](#)

Moody GB, Mark RG. A new method for detecting atrial fibrillation using R-R intervals. *Computers in Cardiology*. 10:227-230 (1983).

CPSC2021:

By accessing and using the HeartX Analyzer project, you agree to be bound by the terms and conditions of the Creative Commons Attribution 4.0 International Public License ("Public License"). If this Public License is interpreted as a contract, you are granted the Licensed Rights in consideration of your acceptance of these terms and conditions. The Licensor grants you these rights in consideration of the benefits the Licensor receives from making the Licensed Material available under these terms and conditions.

[Creative Commons Attribution 4.0 International Public License](#)

Wang, X., Ma, C., Zhang, X., Gao, H., Clifford, G. D., & Liu, C. (2021). Paroxysmal Atrial Fibrillation Events Detection from Dynamic ECG Recordings: The 4th China Physiological Signal Challenge 2021 (version 1.0.0). PhysioNet. <https://doi.org/10.13026/ksya-qw89>.

PhysioNet/CinC Challenge:

[Open Data Commons Attribution License v1.0](#)

Clifford GD, Liu C, Moody B, Li-wei HL, Silva I, Li Q, Johnson AE, Mark RG. AF classification from a short single lead ECG recording: The PhysioNet/computing in cardiology challenge 2017. In 2017 Computing in Cardiology (CinC) 2017 Sep 24 (pp. 1-4). IEEE. <https://doi.org/10.22489/CinC.2017.065-469>

Shaoxing:

By accessing and using the HeartX Analyzer project, you agree to be bound by the terms and conditions of the Creative Commons Attribution 4.0 International Public License ("Public License"). If this Public License is interpreted as a contract, you are granted the Licensed Rights in consideration of your acceptance of these terms and conditions. The Licensor grants you these rights in consideration of the benefits the Licensor receives from making the Licensed Material available under these terms and conditions.

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Copyright (c) 2012-2020 The PyWavelets Developers
Medizin- und Informationstechnik AG