
HeartX[®] Recorder

Quick Start Guide

Deutsch (DE).....	5
English (EN).....	11
Dansk (DA).....	17
Česky (CS).....	23
Ελληνικά (EL).....	29
Español (ES).....	35
Français (FR).....	41
Magyar (HU).....	47
Italiano (IT).....	53
Nederlands (NL).....	59
Norsk (NO).....	65
Polski (PL).....	71
Português (PT).....	77
Svenska (SV).....	83
Türkçe (TR).....	89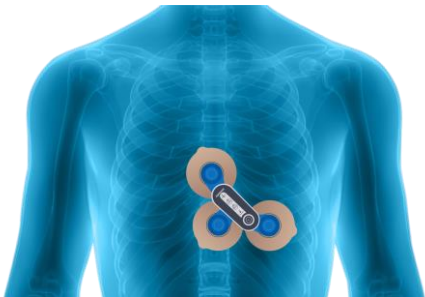

*3

Deutsch (DE)

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English (EN)

To access these and the detailed instructions for use online go to getemed.de/user-manuals and click on **HeartX Recorder**.

To request paper manuals at no charge, contact your sales representative. Allow 7 days for delivery.

1. Device Operation

1.1 Button

- Press and hold until a beep sounds to turn on the device.
- Press and release to start recording.
A double beep confirms that recording has started.

English (EN)

1.2 Prepare the Docking Station

- Connect the USB cable to the port located at the bottom of the docking station.
- Connect the other end to a compatible USB power source → Computer or USB wall adapter

CAUTION

Only connect the docking station to USB power sources that meet the EN IEC 60950 or EN IEC 62368 standards to avoid potential safety hazards and to ensure compliance with regulatory requirements. Using non-compliant power sources may damage the device or lead to unsafe operation.

2. Patient Hookup

- Prepare the patient – shave and clean the skin.
- Attach the electrodes to the device.
- Place the device on the patient's skin.
- Press the button to start recording.
- The device will automatically start the recording after eight minutes when detecting a signal of three channels.

3. Device States via LED

GREEN – ECG quality is good

ORANGE – Lead-off, check position

GREEN – Battery sufficient for 24 hour recording

GREEN FLASHING – Charging

GREEN – Battery sufficient for 7 day recording

GREEN FLASHING – Charging

ORANGE – Recording available

ORANGE FLASHING - Download in progress

English (EN)

4. Cleaning and Disinfection

The following steps must be performed at regular intervals, prior to first use, and before passing the device on to another person.

4.1 Preparation

- Let the device switch off before cleaning and disinfection.

4.2 Cleaning

- Use a lint-free cloth slightly moistened with a mild soap solution to wipe the surface of the device.
- Ensure all visible dirt and debris are removed from the device and especially on the gold-plated contacts before proceeding to disinfection.

4.3 Disinfection

- Disinfect the device using a 70% alcohol solution.
- Ensure to follow the recommended application time for the disinfectant, which is 10 minutes for a 70% alcohol solution.
- After the application time, use a lint-free cloth slightly moistened to remove any residue from the disinfectant.

5. Incident Reporting

A serious incident is a device malfunction that results in death or serious injury, or may lead to death or serious deterioration of health.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

6. Supplies and Accessories

This chapter identifies supplies and accessories that are approved for use with your system.

Part Number	Description
86100	HeartX Recorder
86150	HeartX Recorder Docking Station
86550	USB-C to C data and charging cable
86650-EN	HeartX Recorder Instructions for Use, EN
86660-UNI	HeartX Recorder Quick Start Guide, Multilingual
86501	Patch Typ „F“ (female) *1
86502	Patch Typ „M“ (male) *2
90131	Gel-Elektrodes *3

English (EN)

7. Maintenance and Repair

Only authorized personnel are allowed to repair the device. Any unauthorized attempts to repair the device will make any warranty claims null and void. It is the operator's responsibility to report the need for repair to the manufacturer. If you determine or suppose any malfunction, send the device for checking to the address specified below. Please add a detailed error description. If you determine an unexpected operational condition or unexpected occurrences or if you need technical support, contact the manufacturer under the following address:

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6. Forbrugsstoffer og tilbehør

Dette kapitel beskriver forbrugsstoffer og tilbehør, der er godkendt til brug med dit system. A serious incident is a device malfunction that results in death or serious injury, or may lead to death or serious deterioration of health. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Varenummer	Beskrivelse
86100	HeartX Recorder
86150	HeartX Recorder Docking Station
86550	USB-C to C data and charging cable
86650-DA	HeartX Recorder Instructions for Use, DA
86660-UNI	HeartX Recorder Quick Start Guide, Multilingual
86501	Patch Typ „F“ (female) *1
86502	Patch Typ „M“ (male) *2
90131	Gel-elektroder *3

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GETEMED Medizin- und Informationstechnik AG
Medizin- und Informationstechnik AG